


LRD410108 Armrest Assembly

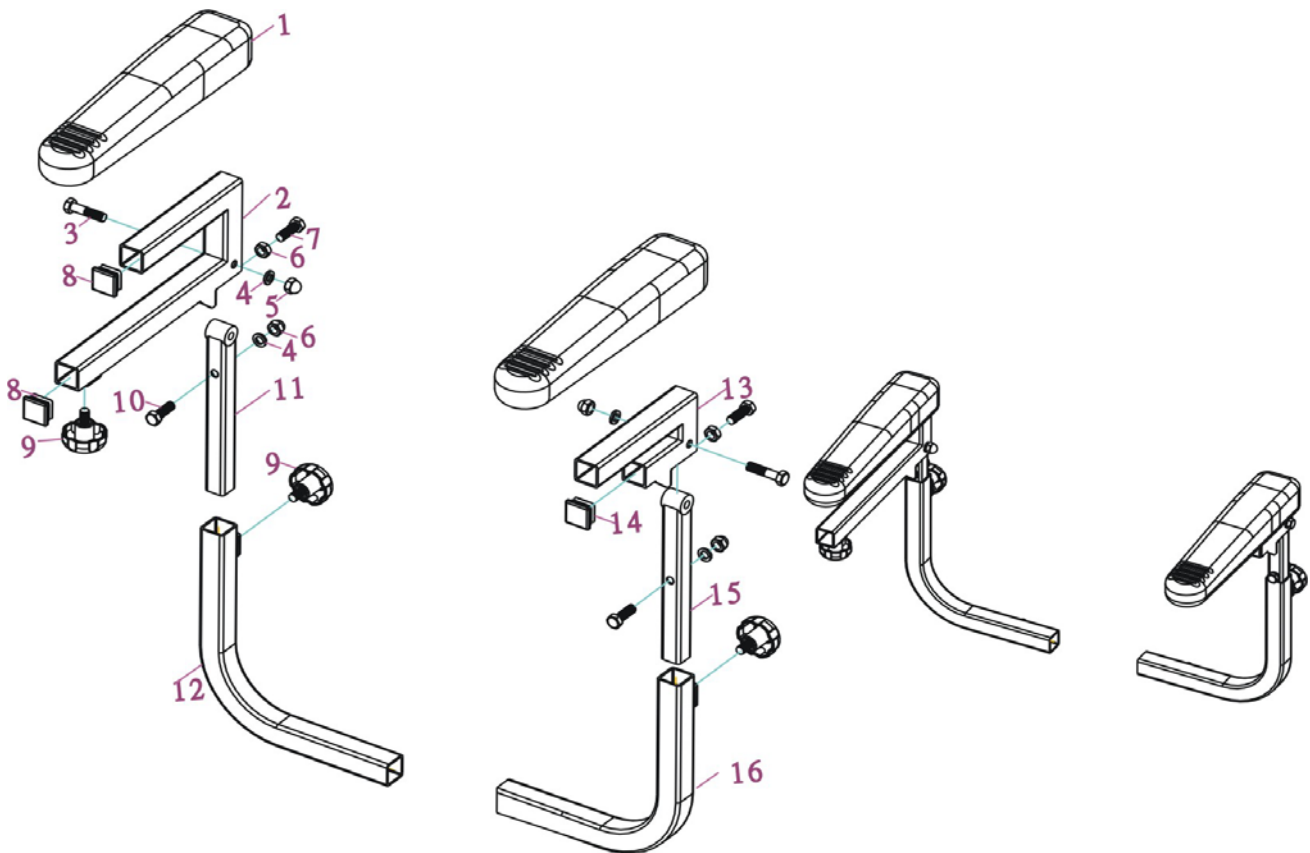

Item	Part Number	Description	Qty.
1	LRD410108-01	Arm Pad	2
2	LRD410108-02	Bracket	2
3	LRD410108-03	Hex Head Cap Screws	2
4	LRD410108-04	Spring Washer	4
5	LRD410108-05	Nut	4
6	LRD410108-06	Nut	2
7	LRD410108-07	Hex Head Cap Screws	2
8	LRD410108-08	Tub Cap	2
9	LRD410108-09	Armrest Screw	4
10	LRD410108-10	Hex Head Cap Screws	2
11	LRD410108-11	Arm Post	1

Item	Part Number	Description	Qty.
12	LRD410108-12	Right Elbow	1
13	LRD410108-13	Arm Pad Bracket	1
14	LRD410108-14	Tub Cap	1
15	LRD410108-15	Arm Post	1
16	LRD410108-16	Left Elbow	1