


LRM302213 Seat Assembly

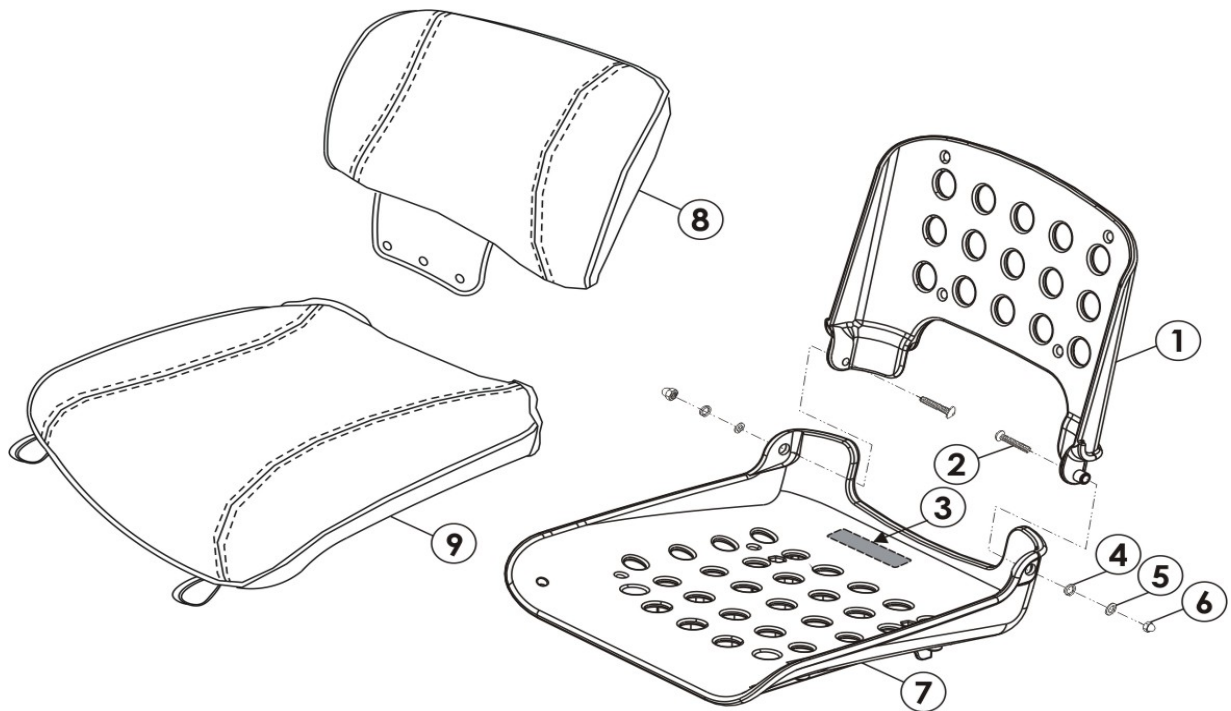

Item	Part Number	Description	Qty.
1	LRM302213-01	Backrest Plastic	1
2	LRM302213-02	Hex Socket Cap Screws	2
3	LRM302213-03	Twin Adhesive (Male)	1
4	LRM302213-04	Washer	2
5	LRM302213-05	Spring Washer	2
6	LRM302213-06	Cap Nut	2
7	LRM302213-07	Seat Plastic	1
8	LRM302213-08	Backrest Pad	1
9	LRM302213-09	Seat Pad	1